# Magnet Venting Conformance Assessment Form

DOC2705036  
Revision 3

# Magnet Venting Conformance Assessment Form

<b>Site Identification:</b>	
Customer/Hospital/Clinic Name:	
Site name:	
Country:	

Requirement	Check Correct Response	
<b>Section 2.2: HVAC Vent Requirements</b>		
Does the incoming air contain at least 5% air from outside the Magnet Room?	<input type="checkbox"/> Yes	<input type="checkbox"/> No
Comment		
Date of Report:		
Completed by (Company name):		
Completed By (First and last name):		
Completed by (Title):		
Signature		

Requirement	Check Correct Response	
<b>Section 2.3: Emergency Exhaust Vent Requirements</b>		
Is the emergency exhaust vent installed at the highest point of the finished/drop ceiling or at the top of the side wall on the coldhead side?	<input type="checkbox"/> Highest point	<input type="checkbox"/> Side wall
Has the emergency exhaust vent been tested and is it operational?	<input type="checkbox"/> Yes	<input type="checkbox"/> No
Do the emergency exhaust fan and the exhaust intake vent each have a capacity of at least 34 m <sup>3</sup> /minute (1200 CFM) with a minimum of 12 room air exchanges per hour?	<input type="checkbox"/> Yes	<input type="checkbox"/> No
Is there a manual exhaust fan switch near the Operator Workspace (OW) and in the Magnet Room near the door?	<input type="checkbox"/> Yes	<input type="checkbox"/> No
Is the exhaust fan activated by an oxygen monitor?	<input type="checkbox"/> Yes	<input type="checkbox"/> No
Is the Magnet Room exhaust fan installed per Figure 2-1 or Figure 2-2 of 5850263-xxx?	<input type="checkbox"/> Yes	<input type="checkbox"/> No
List applicable <b>Figure number</b> .		
Comment		
Date of Report:		
Completed by (Company name):		
Completed By (First and last name):		
Completed by (Title):		
Signature		

Requirement	Check Correct Response	
<b>Section 3.1: Cryogen Venting Requirements</b>		
Does the vent pipe diameter meet the specifications per Table 3-1 of 5850263-xxx where the pipe interfaces with the magnet adaptor?	<input type="checkbox"/> Yes	<input type="checkbox"/> No
Is the vent size designed per the vent pressure drop table listed in section 4 <i>Reference</i> of 5850263-xxx? List applicable <b>Table number</b> .	<input type="checkbox"/> Yes	<input type="checkbox"/> No
<b>Section 3.2: Venting Requirements</b>		
Is the cryogenic vent supported in such a way that it will not transfer any load to the magnet vent adaptor?	<input type="checkbox"/> Yes	<input type="checkbox"/> No
Are the vent options and dimensions selected from Table 3-2 of 5850263-xxx? List the <b>Magnet</b> used from Table 3-2 of 5850263-xxx here.	<input type="checkbox"/> Yes	<input type="checkbox"/> No
<b>Section 3.4: Vent Requirements Inside the Magnet Room</b>		
Is the vent made of one of the materials listed in Table 3-3 of 5850263-xxx? List the <b>Material</b> used from Table 3-3 of 5850263-xxx here.	<input type="checkbox"/> Yes	<input type="checkbox"/> No
Are the tubes or pipes seamless or do they have welded/brazed seams and joints between sections? (Corrugated pipe and/or spiral duct is not permitted)	<input type="checkbox"/> Yes	<input type="checkbox"/> No
Are the vent pipes designed to withstand 241.4 kPa (35 psi)?	<input type="checkbox"/> Yes	<input type="checkbox"/> No
Are the vent supports designed to withstand loads, expansion and contraction during a quench? List the <b>Product</b> from Table 1-1 of 5850263-xxx here.	<input type="checkbox"/> Yes	<input type="checkbox"/> No
Is the vent adaptor a straight adaptor or a 90 degree adaptor? Answer one of the following two questions.	<input type="checkbox"/> Straight	<input type="checkbox"/> 90 degree
<b>For straight adaptor configurations:</b> Is there enough clearance to accommodate at least 102 mm (4 in.) for the magnet vent adaptor, in addition to the required gap of 10 to 32 mm (0.4 to 1.25 in.) per Figures 3-3 and 3-4 of 5850263-xxx?	<input type="checkbox"/> Yes	<input type="checkbox"/> No
<b>For 90 degree adaptor configurations:</b> Is there enough clearance to accommodate the required gap of 10 to 32 mm (0.4 to 1.25 in.) per Figures 3-5 or 3-6 of 5850263-xxx?	<input type="checkbox"/> Yes	<input type="checkbox"/> No
Is only one dielectric break installed in the Magnet Room and is it installed at the magnet interface?	<input type="checkbox"/> Yes	<input type="checkbox"/> No
Is there a minimum clearance of 102 mm (4 in.) between the ceiling or wall both inside and outside the Magnet Room to accommodate the dielectric breaks per Figure 3-7 of 5850263-xxx? NOTE: The overall length of the waveguide must be at least 812 mm (32 in.)	<input type="checkbox"/> Yes	<input type="checkbox"/> No
Comment		
Date of Report:		
Completed by (Company name):		
Completed By (First and last name):		
Completed by (Title):		
Signature		

Requirement	Check Correct Response	
<b>Section 3.5: Vent Requirements Outside the Magnet Room</b>		
Are the tubes or pipes seamless or do they have welded/brazed seams and joints between sections? (Corrugated pipe and/or spiral duct is not permitted)	<input type="checkbox"/> Yes	<input type="checkbox"/> No
Are the vent supports designed to withstand loads, expansion and contraction during a quench?	<input type="checkbox"/> Yes	<input type="checkbox"/> No
List the <b>Product</b> used from Table 1-1 of 5850263-xxx here.		
Is only one dielectric break installed outside the Magnet Room, adjacent to the wave guide, and is there a 19 to 32 mm (0.75 to 1.25 in.) gap to the vent pipe per Figure 3-8 of 5850263-xxx?	<input type="checkbox"/> Yes	<input type="checkbox"/> No
Are any electromechanical fire dampers used?	<input type="checkbox"/> Yes	<input type="checkbox"/> No
Is condensate prevented from pooling inside any section of the venting system by use of downward tilt or weep holes?	<input type="checkbox"/> Yes	<input type="checkbox"/> No
Does the vent exhaust exit have an exclusion area per Figures 3-9, 3-10 and 3-11 of 5850263-xxx?	<input type="checkbox"/> Yes	<input type="checkbox"/> No
Is the vent exit area free of air intake vents and windows to prevent cryogen exhaust from reentering the facility per Figures 3-9, 3-10 and 3-11 of 5850263-xxx?	<input type="checkbox"/> Yes	<input type="checkbox"/> No
Is the vent outlet covered with square screen mesh and does it have a cover/cap over the pipe, if the vent outlet is orientated in the vertical up position?	<input type="checkbox"/> Yes	<input type="checkbox"/> No
Was the exhaust vent cover or cap included in the pressure drop calculation?	<input type="checkbox"/> Yes	<input type="checkbox"/> No
Combined cryogen venting from two GE HealthCare MR systems must be noted and can be used if construction follows this requirement with separation plates, supports and properly designed pressure drops per Figure 3-12 of 5850263-xxx. Is there a combined vent?	<input type="checkbox"/> Yes	<input type="checkbox"/> No
Are all other requirements contained within the Magnet Room Venting manual understood and adhered to?	<input type="checkbox"/> Yes	<input type="checkbox"/> No
Comment		
Date of Report:		
Completed by (Company name):		
Completed By (First and last name):		
Completed by (Title):		
Signature		

Site Identification:	To be filled out by GE HealthCare
SYSID/GON:	
Type of GE HealthCare system (Product name):	
Type of GE HealthCare magnet:	
GE HealthCare Concession (if applied):	
Date Received:	
Received By (Name and SSO):	
Received by (Title):	

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